D

ReCord®

Gebrauchsinformation

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ReCord®

Instruction of use

Knitted retraction cord description:

Knitted retraction cord is made of 100% cotton, knitted into thousands of tiny loops forming long interlocking chains.

The unique knitted design exerts a gentle continuous outward force following placement as knitted loops seek to open. The cord is designed to achieve better tissue displacement and retraction, as unique design is easy for placement and stay put better in sulcus than conventional cord. Also knitted retraction cords are better in terms of hemostasis effectiveness, since they carry significantly greater quantities of hemostatic solution than conventional cords.

Instruction of Use for Retraction Cord

Important note: Knitted retraction cord condenses when packed; use a size larger than expected.

Steps:

1. Select an appropriate size cord. The knitted cord is designed to compress upon packing, pinching tightly the sulcus. Use a size cord, which appears larger than with braided cords, as the knitted cord, compresses smaller upon packing.
2. Often the size#2 and occasionally the #3 are use with molars. The #1 is often used for bicuspid and upper. The #0 or #00 is often used for lower anteriors. The #000 is designed as a bottom cord for those who prefer to pack two cords, leaving the smaller cord in place while delivering the impression material. Tissue firmness and friability, as well as the amount of space desired may require using different sizes than what is suggested here. The size #000 is used at preparation or at the cementation of veneers and everywhere a close-fitting sulcus-tissue is presented. Also this size is used as lower thread at the double-end technique. If needed or preferred, soak the knitted cord in hemostatic /displacement solution to assist the gingival tissue displacement. (Knitted retraction cord is suitable to be used in combination with hemostatic solutions and displacement solutions such as aluminum chloride or alum solutions or impregnated with epinephrine hydrochloride.)
3. Use retraction cord packer to place the cord into the sulcus.
4. Following cord placement, thoroughly rinse the area, optimal tissue displacement effect occurs in 1-3 minutes.
5. Remove cord from sulcus before making impression. Again give a film air/water spray to clean preparation and check for effectiveness of hemostasis.

Knitted Retraction Cord Sizes:

- REF** WP4921, Size #000
- Preparing and cementing veneers
 - Restorative procedures dealing with thin, friable tissues
- REF** WP4915, Size #00 / **REF** WP4916, Size #0
- Lower anteriors
 - When luting near gingival and subgingival veneers
 - Class III, IV, and V restorations
 - Second cord for "two-cord" technique
- REF** WP4917, Size #1
- Size 1 and Size 2 are particularly effective for tissue control and/or displacement when soaked in coagulative hemostatic solution prior to and/or after crown preparations
 - Use as a protective "pre-preparation" cord on anteriors
- REF** WP4918, Size #2
- Upper cord for "two-cord" technique
 - Use as a protective "pre-preparation" cord on anteriors
- REF** WP4919, Set with each 1x Size #00, #0, #1, #2

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Willmann & Pein GmbH