D

Securacem®

Gebrauchsinformation

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Securacem®

Instructions for use

Securacem ist ein X-ray opaque glass-ionomer cement that has been specifically developed as an underfilling material underneath composite and amalgam fillings. Securacem increases the biocompatibility of composites and improves the border seal. Adhesion to the dentin and enamel is by way of the reaction of the carboxylic groups of polyacrylic acid with the calcium or hydroxylapatite and collagen. This direct chemical bonding ensures that far more healthy dental material is saved because undercutting is not necessary.

Classification
DIN EN ISO 9917-1:2004-04 requirements are fulfilled

Preparing the cavities
A maximum of healthy dentine must be retained when preparing the cavities. The cavity must be shaped in such a manner that the cement layer is not thinner than 0.5 mm at any point.

Pulp protection
Rinse the prepared cavities thoroughly with water and dry with oil-free blown air. Use calcium hydroxide preparations as an pulp-capping for very deep cavities.

Preparing the dentin
Clean the freshly prepared dentin surface, rinse thoroughly with water and dry with blown air. Avoid dehydration.

Mixing Securacem
Shake powder well before use! Use one level dosing scoop for two full drop of liquid (without air bubbles; hold the bottle vertically downwards and apply light pressure to squeeze out one drop). First mix only half the amount of powder, then continue with the other half of the remaining powder until the required consistency is achieved. Use only a small area of the mixing block in order to minimize water evaporation.

Application of Securacem
The mixed material is lightly pressed into the cavity. Securacem is workable for 1'40" minutes (incl. mixing). The material should not be handled any more when hardening sets in. The hardening time is 3'00"-3'40" minutes.

Paket içeriği
REF WP4072 15g powder / 10g liquid + dosing scoop
REF WP4092 7g powder / 5g liquid + dosing scoop
REF WP4147 10g liquid

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