Dent-a-cav[®]

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Page 3 - en: Instructions for use

Page 4 - fr: Manuel d'instruction

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Strana 7 - bs: Uputstvo za upotrebu

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data:

Calcium sulphate, zinc oxide, ethyl cellulose

Quantitative data:

60 % calcium sulphate

15 % zinc oxide

Intended purpose of the product

For temporary fillings.

Indications

Temporary, short-term closure of dental cavities.

Contraindications

None

Performance characteristics of the product

Sealing compound, self-curing.

Specific precautions

None

Application

Apply the required amount of Dent-a-cav into the moist cavity with a filling instrument. Hardening begins after a few minutes. In exposed areas, a 15-minute rest is recommended. Deep cavities do not need to be filled completely, thereby facilitating later removal. When closing inserts, ensure that the application is as free of pressure as possible.

The best way to remove the set Dent-a-cav is to use a pointed, short and strong probe inserted between the dentin and the occlusion.

Warnings**Residual risks**

The processing instructions and precautionary measures described in the instructions for use accompanying the product must always be observed. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

Side effects

Adverse side effects are not known. Risks in case of overdose are not known.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

Close the container immediately after each material withdrawal.

Shelf life of the closed packaging: see label.

After opening for the first time, the preparation can be stored for at least 6 months. The Product should be stored between 0° C - 22°C.

Minimum shelf life

Do not use Dent-a-cav after the expiry date.

Note on Dent-a-cav pink:

Colour deviations from pink do not represent a defect and have no influence on the function, usability and safety of the product.

Variants

REFWP4311 2 glasses at 30.0 g, pink

REFWP4312 2 glasses at 30.0 g, white

REFWP5627 1 glass at 30.0 g, pink

REFWP5628 1 glass at 30.0 g, white

Disposal

Unused material can be disposed of with household waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Attention **Warning**

H315 causes skin irritation
H319 causes serious eye irritation
H411 toxic to aquatic organisms, with long-lasting effect

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