# WP PerfectPrint Model®

## für Modelle

### Gebrauchsinformation

#### Allgemeine Informationen

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### Instructions for Use

#### General information

WP PerfectPrint Model is a methacrylate-photoinitiators, pigments and additives. based light-curing resin for use in the dental laboratory. For DLP 3D printers operating at a wavelength of 385-405 nm.

#### Intended use

Production of dental models for 3D printing technology.

#### Special precautions

- Non-compliance with instructions and improper use may lead to impairment of quality.
- Wear protective gloves, protective clothing and safety goggles while working.
- Avoid contact with eyes! If liquid material comes into contact with eyes inadvertently, rinse immediately with plenty of water and consult a doctor if necessary.
- Avoid direct skin contact with liquid material and components prior to post-curing.
- Keep out of reach of children.
- Observe safety data sheet.

#### Side effects

- Irritation of eyes and skin.
- The polymerised material should not be used if there are allergies to any of the ingredients or in case of contact allergies.
- In rare cases, allergic reactions to components of WP PerfectPrint A cXY may occur. Inhalation of monomer vapour and skin contact with non-polymerised materials should be avoided.

#### Notes on use

- Shake the bottle for at least one minute before use.
- When filling the pan with resin, observe the instructions of the device manufacturer.
- Note the temperature that the device manufacturer specifies for production. The resin can also be processed at room temperature.
- Darken the room slightly before removing the print platform with the printed object. Then, open the hood and remove the printing platform with the printed object.
- Remove the printed object from the platform.
- Only remove the support structures after cleaning and post-curing.

#### Using used resin

- After sieving (removal of polymerised particles) with a sieve gauze (about 200 µm), the used resin can be mixed with fresh resin 1:1 and reused.

#### Post-processing of the printed object

- Clean the printed object with ethanol or with a soft brush for a maximum of 3 minutes.
- Optional: Clean printed object in a separate container with fresh ethanol in an ultrasonic bath as needed (5-10 mins).
- Optional: Clean openings, holes and gaps with compressed air.

#### Post-exposure of the printed object

It is recommended to post-cure the printed object on each side for 20 minutes in a UVA light-curing device with 4 lamps of 9 watts each. Please note the information provided by the device manufacturer.

#### Restrictions on use

- Intended exclusively for the creation of dental models in the dental sector.
- Must not be brought into direct contact with the patient.

#### Composition

Mixture of acrylate and methacrylate resins, photoinitiators, pigments and additives.

#### Storage and durability

- Store at room temperature/0-22 °C in a dark, dry place.
- Only handle WP PerfectPrint A cXY as briefly as possible to reduce exposure to daylight/room light. Excessive exposure to light leads to an unwanted hardening of the material.
- Close the container immediately after each use and keep it tightly closed.
- Do not use after the expiration date!

#### Packaging

- REF WP5171 WP PerfectPrint A cXY, grey, 1000 g
- REF WP5172 WP PerfectPrint A cXY, grey, 500 g
- REF WP5175 WP PerfectPrint A cXY, beige, 1000 g
- REF WP5176 WP PerfectPrint A cXY, beige, 500 g
- REF WP5173 WP PerfectPrint A cXY, yellow, 1000 g
- REF WP5174 WP PerfectPrint A cXY, yellow, 500 g
- REF WP5186 WP PerfectPrint A cXY, pink, 1000 g
- REF WP5187 WP PerfectPrint A cXY, pink, 500 g

**dent a pharm**  
Produktionsgesellschaft mbH  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9225-0  
Fax: +(49) 4123/9228-49  
www.dent-a-pharm.de  
info@dent-a-pharm.de

**Willmann & Pein GmbH**  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9228-0  
Fax: +(49) 4123/9228-49  
www.wp-dental.de  
info@wp-dental.de

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