# Calcident 450<sup>®</sup>

Seite 2 - de: Gebrauchsanweisung  
Page 3 - en: Instructions for use  
Page 4 - fr: Manuel d'instruction  
Página 5 - es: Instrucciones de uso  
Pagina 6 - it: Istruzioni per l'uso  
Strana 7 - bs: Uputstvo za upotrebu  
Pagină 8 - ro: Instrucțiuni de utilizare  
Sayfa 9 - tr: Kullanma kılavuzu  
Стр. 10 - ru: Инструкция по применению

*We know how*

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**Patientenzielgruppe**

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## Instructions for use

**Patient target group**

Children, adolescents, adults and older patients

**Intended users**

Dentist

**Composition**

Qualitative information according to the information in the biocompatibility:

Calcium hydroxide, water, barium sulphate, hydroxyethyl cellulose

Quantitative data: Substances

(Quantity according to biocompatibility):

45 % calcium hydroxide,

5 % barium sulphate

**Intended purpose of the product**

Calcident 450 is an aqueous calcium hydroxide paste.

**Indications**

Treatment of caries-free dentin close to the pulp, deep caries treatment, indirect pulp capping, direct pulp capping, pulpotomy, and temporary insertion in the root canal.

**Contraindications**

The use of Calcident 450 is contraindicated: in case of pulp opening/ exposure in carious dentin, in case of inflammatory pulp changes, in case of hypersensitivity to one of the ingredients.

**Performance characteristics of the product**

Alkalisiation through calcium hydroxide

**Specific precautions**

None

**Application**

The amount of Calcident 450 to be administered depends on the respective application:

1. Deep caries treatment / indirect pulp capping - Calcident 450 is applied to the cavity floor as a thin film without pressure. A temporary or final filling is then placed.
2. Direct pulp capping - After the working area has been dried, Calcident 450 is applied and a suitable covering filling is then placed to seal the tooth permanently.
3. Vital amputation - Calcident 450 creates a favourable condition for healing of the remaining pulpal stumps due to the recognised localised corrosive effect of calcium hydroxide on the residual pulp.
4. Underfilling / pulp protection - The space-saving radiopaque Calcident 450 also forms a thin whitish insulating layer against cement acids or free monomer in the anterior region under silicate or resin fillings as pulp protection.
5. Dentin wound dressing for ground vital teeth - Calcident 450 is applied to the prepared tooth stump with a brush or cotton swab and carefully dried with warm air. Avoid contact with the gingiva. The brush should first be dipped in boiled water or distilled water to dilute the paste slightly.
6. Medical insertion into infected root canals. Each root canal should be filled with Calcident 450 without air bubbles. If necessary, the insert must be renewed after 6 to 8 weeks.

**Warnings****Residual risks**

Calcident 450 must always be kept tightly closed.

Do not apply increased pressure to the opened pulp during application. Calcium hydroxide causes local irritation and corrosive effects on skin, eyes and mucous membrane. Calcium hydroxide particles on the cornea and in contact with the eye must be removed immediately by suitable mechanical means, then rinse immediately and thoroughly with water.

This medical device must not be used after the expiry date.

This medical device must be kept out of the reach of children.

**Side effects**

Adverse side effects are not known. Risks in case of overdose are not known.

**Interactions**

Interaction of the product with other medical devices, medicinal products or other substances is not known.

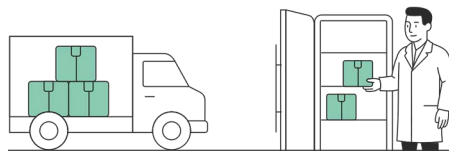
**Storage****IMPORTANT NOTICE:**

The temperature range of 15°C to 23°C indicated on the product packaging applies only during transport.

Upon receipt, the product must be immediately stored under refrigerated conditions at +2°C to +8°C.

🔥 15–23°C

❄️ 2–8°C

**Minimum shelf life**

Do not use after the expiry date.

**Variants**

REF WP4200 1 x syringe with 2.0 ml paste + cannulas

REF WP4201 2 x syringes with 2.0 ml paste + cannulas

**Disposal**

Unused material can be disposed of with household waste.

**All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.**

*We know how*

**dent a pharm**  
Produktionsgesellschaft mbH  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9225-0  
Fax: +(49) 4123/9225-48  
www.dent-a-pharm.de  
info@dent-a-pharm.de

**Willmann & Pein GmbH**  
Schusterring 35  
25355 Barmstedt/Germany  
Fon: +(49) 4123/9228-0  
Fax: +(49) 4123/9228-49  
www.wp-dental.de  
info@wp-dental.de

Made in Germany

H318 Causes serious eye damage  
H335 May cause respiratory irritation

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