Glass Liner®

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Strona 10 - pl: Instrukcja użytkowania
Стр. 11 - ru: Инструкция по применению

We know how

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Instructions for use

Intended purpose

Glass Liner is a light-curing dental medical product for professional intraoral application as a cavity liner and underfilling material. It can be used by the dentist as a part of restorative treatments on any patient, without restrictions in terms of age or gender.

Indication / Areas of application

Indicated for lining prepared cavities under direct restorations with filling materials.

Contraindications

The material contains methacrylate and BHT. Do not use on patients with known allergies to the ingredients. Application involving direct or indirect contact with the pulp is not indicated.

Side effects

No side effects are known to date.

Residual risks

The residual risk of hypersensitivity to the material components cannot be ruled out.

Interactions

Phenolic substances, in particular compounds containing eugenol and thymol, interfere with curing of composites.

Target group

All patients, without restrictions in terms of age or gender.

Intended user

It is applied by a user with professional dentistry training.

Application of the treatment

1. Clean the tooth before preparation and remove all cleaning residues. Ensure that the tooth surfaces are completely dry and avoid contamination.
2. Protect the tooth pulp.
3. Etch the cavity using the preferred technique (e.g. Total-Etch). Follow the manufacturer's instructions for the materials used for this.
4. Apply bonding agent in accordance with the manufacturer's instructions.
5. Shortly before application, place the amount of the material required on a mixing block. Apply a thin layer of the lining material with an application tool. The layer thickness should not exceed 1 mm. If a thicker layer is required, work in layers and allow them to cure in succession. Remove excess material and then cure with a light curing device (at 450 nm with a minimum light intensity of 400 mW/cm²) at a distance of approx. 2 mm from the surface of the lining for at least 30 seconds.
6. Any stubborn residues on the tooth enamel should be removed with a rotating instrument.
7. Apply the filling material in accordance with the manufacturer's instructions.

Warnings and precautions:

If the dentine over the pulp is less than 0.5 mm (pulp showing through), a pulp protection compound (calcium hydroxide) must always be used to neutralise any clinically relevant diffusion of plastic components that may occur.

Protect from daylight and operating light during application to prevent premature curing of the material.

Avoid contact with the skin. In the event of accidental skin contact, immediately wash the area in question thoroughly with soap and water.

Avoid contact with eyes. In event of accidental eye contact, rinse immediately with plenty of water and seek medical advice if necessary.

Composition

Qualitative composition: glass ceramic, glass ionomer cement, silica, camphorquinone, hexanediol dimethacrylate, Bis-GMA, BHT, DMTBA

Quantitative composition: Fillers 65%, activators, accelerators and stabilisers 1%, dimethacrylate 34%

Storage / Disposal

Store at 0°C - 22°C, protected from light and tightly sealed..

Before application, the material must be brought to room temperature.

Do not use after the expiry date.

Dispose of in accordance with national regulations.

Variants

 WP5601 1 x syringe containing 2.0 ml + needles

 WP5602 2 x syringe containing 2.0 ml + needles

 WP5680 Glass Liner (TR) 1 x syringe containing 2.0 ml + needles

 WP5681 Glass Liner (TR) 2 x syringe containing 2.0 ml + needles

Reporting obligation

Any serious incident that occurs in connection with the product, such as the death or temporary or permanent deterioration in the health condition of a patient, user or other person, and any serious risk to public health that occurs or could have occurred in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is based.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany

Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48

www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany

Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49

www.wp-dental.de
info@wp-dental.de

H315 Causes skin irritation
H335 May cause respiratory irritation
H319 Causes serious eye irritation

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