## Gebrauchsinformation

## Instructions for use

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For professional use only.

## Product description

Pro Crown®, is a highly esthetic composite for fabricating temporary restorations. It has similar properties to a filling material. Pro Crown® temporary restorations can also be adjusted and customized using flowable light-curing composites.

## Indications for use

• Pro Crown is a self-curing or dual-curing composite for the fabrication of temporary crowns and bridges, inlays, onlays and veneers.

• Customization of prefabricated acrylic and metal crowns

• Fabrication of long-term temporary restorations

## Warnings

Do not use if there is a known allergy to methacrylic acid esters.

Allergic reactions to the product cannot be ruled out with sensitive persons.

Sensitization possible through skin contact.

Pro Crown® temporary restorations are intended for use as temporary restorations and not as permanent restorations.

Remains of methacrylate-based materials may change the setting properties of silicone impression and bite registration materials. If a restoration has been fabricated previously, prior to impression-taking clean the preparation and surrounding tissue carefully with gauze soaked in alcohol and spray with water.

Temporary luting cements that contain eugenol can inhibit the curing of permanent composite cements, which may be used later for final cementation.

## Cautions

Do not use after expiration date.

Do not leave any residue of the temporary crown and bridge material in the sulcus/on the tooth or oral cavity.

Do not swallow impression material! If swallowed: Consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: Rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

Do not inhale grinding dust. Use safety glasses and face mask!

When taking impressions of areas with severe undercuts and wide interdental spaces, standard blocking-out techniques must be used.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

## Technical data

Working and setting times  
Working-time: 60 sec., Curing/Setting-time: 5 min.

**Note**  
Working times apply for products, which have been stored and processed at a temperature of 23°C/ 74°F and a relative humidity of 50%. Higher temperatures shorten the working time and lower temperatures extend the working time.

## Anatomical impressions

All standard impression materials can be used for fabrication of a temporary crown, bridge, inlay or onlay. Ideal Medium® is ideal for taking an anatomical impression. As an A-silicone if remains dimensionally stable for several weeks. If a temporary restoration requires remaking, the Ideal Medium® impression can be reused.

## Instructions for use

## Preparation

First take an anatomical impression of the dentition. A laboratory fabricated, vacuum-formed splint or, in the case of single crowns, a prefabricated metal or plastic crown can be used instead of an impression for forming the shape.

The impression can be cut out before use to remove any undercuts and increase the stability of the temporary restoration. A channel can be cut into the impression to create a bar-shaped connection between spaced abutment teeth. Denture teeth can also be used as space maintainers in the anterior region before taking the impression.

## Dispensing and mixing

Dispense and mix the pastes using a Mixing-gun (DS-50 4:1/10:1) dispensing gun and corresponding mixing tips. Only use the included mixing tips!

## Initial dispensing

• Insert the cartridge into the Mixing-gun (DS-50 4:1/10:1) dispensing gun

• Remove the cartridge cap.

• Before filling a mixing tip for the first time, advance the plunger into the cartridge and extrude a small amount of material until both materials are dispensed uniformly. Then discard the dispensed paste.

## Application

To ensure optimal results we recommend using vinyl or latex gloves. Place the mixing tip on the double syringe, turn the cap clockwise by 90° until it engages. With the mixing tip in place, the material requires a certain time before flowing out. The material stops flowing immediately once the pressure on the dispenser handle is reduced.

Do not use force to press out any material that has set in the mixing tip, as it may damage the cartridge and mixing tip!

Compomer, resin-modified glass ionomers or composite preparation surfaces must be isolated before application of Pro Crown®, e.g. using Vaseline.

• Extrude an approximately pea-sized amount of material before each use and discard.

• Fill the dried impression, vacuum-formed splint or preformed crown, beginning at the base. Leave the mixing tip immersed in the material during filling to avoid trapping air bubbles.

• The filled mold must be placed in the mouth using light pressure within 45 sec.

• The material attains an elastic consistency in the mouth 2 min. after the beginning of mixing and should be removed from the mouth together with the impression or vacuum-formed splint by 3 min. after the beginning of mixing.

• Check the condition of the material in the mouth using the excess material, e.g. with a probe, to avoid it curing too much before removal.

• The mixing tip should be left on the cartridge after use.

• Before using the cartridge again, remove the filled mixing tip and check the cartridge outlet openings to ensure that no polymerized materials have sealed the openings. Clear any clogged openings and dispense a small amount of material until equal amounts are extruded at the same rate.

• Install a new mixing tip and continue as described above.

**Finishing**

• Completely wipe off the inhibition layer after the material has fully cured, e.g. with alcohol.

• Once the material has cured fully, finish the temporary restoration, e.g. using cross-cut burs and polishers.

## Cleaning/Disinfection

• If required, Pro Crown® can be disinfected using standard aqueous disinfectant solutions. Customization/ Recontouring

Basically, the use of a bonding agent is not necessary when using a filling composite in combination with Pro Crown® for customization and adjusting the shape.

Please adhere to the respective instructions for use under other products.

## Cementation

• Remove any residual separating agent from the prepared tooth before temporary cementation.

• Standard temporary luting materials can be used for cementation.

• Residue of temporary cements that contain eugenol can impair the curing of composite cements used for subsequent permanent luting. If the plan is to use composite cements, the use of a eugenol-free temporary cement is recommended.

## After cementation

• Carefully check the sulcus and surrounding areas and remove any remaining residue of the temporary material or cement.

## Repair

• Roughen the fracture site and create mechanical retention on the adjoining sections.

• Clean using alcohol and then air dry.

• Join or add to newly fabricated restorations using Pro Crown® or a flowable, light-curing filling composite.

• Join or add to existing restorations using Pro Crown® or the combination of a flowable, light-curing filling composite and a bonding agent.

• Cure Pro Crown® or composite according to the respective instructions for use.

• Prepare in the usual way after curing.

## Storage conditions

Dry, protected against light, at room temperature MLX 2.5-12-S 4:1/10:1 prevent extreme temperature ranges. Do not store in fridge.

## Disposal

Dispose of the product in accordance with local regulations.

**Cleaning and disinfection of the Mixing-gun**

• Remove the cartridge

• Remove any uncured paste using a cloth soaked in alcohol.

• The Mixing-gun can be used several times if properly disinfected. Willmann & Pein GmbH recommends exchanging the gun if there are visible traces of use.

Disinfection can be carried out using a commercially available dipping disinfectant. Willmann & Pein GmbH recommends the use of glutaraldehyde-based solutions.

## Packaging:

WP4367, A1, 1x 50ml + Canulas  
WP4356, A2, 1x 50ml + Canulas  
WP4358, A3, 1x 50ml + Canulas  
WP4098, Bleach, 1x 50ml + Canulas

**dent a pharm Produktionsges. mbH**  
Schusterring 35, 25355 Barmstedt/Germany

**Willmann & Pein GmbH**  
Schusterring 35, 25355 Barmstedt/Germany  
Fon: +49-4123-9228-0, Fax: +49-4123-9228-49  
info@wp-dental.de, www.wp-dental.de