Pro Crown®

Seite 2	- de: Gebrauchsanweisung
Page 3	- en: Instructions for use
Page 4	- fr: Manuel d'instruction
Página 5	- es: Instrucciones de uso
Pagina 6	- it: Istruzioni per l'uso
Strana 7	- bs: Uputstvo za upotrebu
Pagină 8	- ro: Instrucțiuni de utilizare
Sayfa 9	- tr: Kullanma kılavuzu
Strona 10	- pl: Instrukcja użytkowania
Стр. 11	- ru: Инструкция по применению
Page 12	- Glossar

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist

Composition

Qualitative data:

Ethoxylated (2) bis-phenol A dimethacrylate, aliphatic urethane dimethacrylate

Quantitative data in % G/G:

Methacrylic acid compounds 56 %, fillers 43 %, activators 1 %

Intended purpose of the product

Pro Crown® is a self-curing composite for the dental fabrication of temporary crowns and bridges, inlays, onlays and veneers on children, adolescents, adults and elderly patients by a dentist.

Indications

Pro Crown® is a self-curing composite for the fabrication of temporary crowns and bridges, inlays, onlays and veneers.

Contraindications

Do not use in case of known allergies to methacrylic acid esters.

Pro Crown® temporary restorations are intended for temporary use and are not to be understood as definitive restorations.

Performance characteristics of the product

Protection of the ground teeth from heat, cold and changes until the final work is inserted.

Clinical benefit

Accurate fit and therefore a better fit of the final crowns or bridges.

Specific precautions

None

ApplicationPreparation

Situation impression:

All standard impression materials can be used to create a temporary crown, bridge, inlay or onlay. Instead of an impression, a laboratory-fabricated, thermoformed splint or, in the case of single crowns, a prefabricated, temporary metal or plastic crown can be used for the impression. The impression can be cut out before use to eliminate possible undercuts and increase the stability of the temporary restoration. A groove can be cut into the impression for the bar-shaped connection of abutment teeth in the case of gaps. In the anterior region, prefabricated teeth can also be used as placeholders before the impression is taken.

Dispensing and mixing

The pastes are mixed and dispensed using a 10 ml double-chamber syringe and corresponding 10:1 mixing cannulas. Only use the enclosed mixing cannulas!

Application

- Remove the cap from the double-chamber syringe.
- Push the plunger into the double-chamber syringe and dispense a small amount of material until both materials are conveyed evenly. Then discard the dispensed paste.
- Place the mixing cannula on the double-chamber syringe, and turn the cap 90° clockwise until it clicks into place.
- Squeeze out the material until the mixed material comes out of the mixing cannula. The material needs a certain amount of time to flow out when the mixing cannula is attached. The material flow is interrupted as soon as the pressure on the plunger decreases.
- Remove a pea-sized amount of material and discard. After that, continue to squeeze out the material and apply it. Die surfaces made of compomers, resin-modified glass ionomers or composites must be isolated before applying Pro Crown®, e.g. with petroleum jelly.
- Fill the dried impression, thermoforming tray or prefabricated crown starting from the bottom. Keep the tip of the mixing cannula immersed in the material to prevent air bubbles from being trapped.
- The filled mould must be inserted into the mouth with light pressure within 45 seconds.
- 2 minutes after the start of mixing, the material reaches an elastic consistency in the mouth and must be removed from the mouth 3 minutes after the start of mixing with the impression or thermoforming tray.
- Check the condition of the material based on the excess, e.g. with a probe in the mouth, to avoid excessive hardening before removal.
- Do not remove the mixing cannula after use – it serves as a seal. Before using the double-chamber syringe again, remove the filled mixing cannula and check the outlet openings of the double-chamber syringe to ensure that no polymerised material is blocking the openings. Unblock any blocked openings and spread a small amount of material until both materials are conveyed evenly.
- Attach a new mixing cannula and continue as described above.

Preparation

- Completely remove the inhibition layer after the material has fully cured, e.g. with alcohol.
- After complete curing, the temporary restoration can be finished, e.g. with cross-cut milling and polishing.

Cleaning

- If necessary, Pro Crown® can be cleaned with the usual cleaning agents.

Individualisation/form correction

When using a filling composite in combination with Pro Crown®, the use of a bonding agent is generally not necessary for individualisation or form correction. For other products, please refer to the respective instructions for use.

Cementation

- Remove any insulation residue on the die before temporary fixation.
- Standard temporary bonding materials can be used for cementation.
- Residues of temporary cements containing eugenol may impair the curing of composite cements for subsequent permanent cementation. The use of an eugenol-free temporary cement is therefore recommended for the planned use of composite cements.

After cementation

- Carefully check the sulcus and the surrounding areas and remove any remaining temporary material or cement.

Repair

- Roughen the fracture site and create mechanical retention in the neighbouring areas.
- Clean with alcohol and then dry with air.
- Join or supplement newly fabricated restorations with Pro Crown® or a flowable, light-curing restorative composite.
- Join or supplement existing restorations with Pro Crown® or the combination of a flowable, light-curing restorative composite and a bonding agent.
- Pro Crown® or cure the composite according to the corresponding instructions for use.
- After hardening, finish as usual.

Technical data

Processing and setting times, processing time: approx. 60 sec., setting time: approx. 5 min.

Note

The processing times apply to products that have been stored and processed at a room temperature of 22 °C-23 °C and 50 % relative humidity.

Higher temperatures shorten and lower temperatures lengthen these times.

Warnings**Residual risks**

Do not use in case of known allergies to methacrylic acid esters. Allergic reactions to the product cannot be ruled out in sensitive persons. May cause sensitisation by skin contact. Pro Crown® temporary restorations are intended for temporary use and are not to be understood as definitive restorations. Residues of methacrylate-based materials can change the setting behaviour of silicone impression and bite registration materials. If a restoration was fabricated previously, carefully clean the preparation and surrounding tissue with alcohol-soaked gauze and water spray before taking the impression. Temporary bonding cements containing eugenol can inhibit the setting of permanent composite cements, which may be used later for permanent bonding.

May cause sensitisation by skin contact.

Side effects

No undesirable side effects are known.

Interactions

The product is not known to interact with other medical devices.

Storage

The Product should be stored between 0° C - 22°C. Store in a dry place, protected from sunlight, avoid extreme temperatures. Do not store in the refrigerator.

Minimum shelf life

Do not use after the expiry date.

Variants

[REF]WP4089, A1, 1x 10 ml double-chamber syringe + mixing cannulas

[REF]WP4046, A2, 1x 10 ml double-chamber syringe + mixing cannulas

[REF]WP4357, A3, 1x 10 ml double-chamber syringe + mixing cannulas

[REF]WP4097, bleach, 1x 10 ml double chamber syringe + mixing cannulas

Disposal

Hardened residues can be disposed of with household waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

H315 Causes skin irritation
H319 Causes serious eye irritation
H317 May cause an allergic skin reaction

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