Competence universal[®]

Seite 2	- de: Gebrauchsanweisung
Page 3	- en: Instructions for use
Page 4	- fr: Manuel d'instruction
Página 5	- es: Instrucciones de uso
Pagina 6	- it: Istruzioni per l'uso
Strana 7	- bs: Uputstvo za upotrebu
Pagină 8	- ro: Instrucțiuni de utilizare
Sayfa 9	- tr: Kullanma kılavuzu
Strona 10	- pl: Instrukcja użytkowania
Стр. 11	- ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist

Composition

Qualitative data:

Oligomeric methacrylates, silicon dioxide, glass ceramics

Quantitative data in % G/G:

Oligomeric methacrylates 23, fillers midi 4, mini 65 and nano 8

Intended purpose of the product

Competence Universal is a light-curing, radiopaque, composite filling material for the dental restoration of Class I to V cavities in children, adolescents, adults and elderly patients by a dentist.

Indications

Indicated for direct filling and restoration of Black Class I-V tooth structure defects in anterior and posterior teeth. The material is used for functional and aesthetic reconstruction of natural tooth shapes by means of light-curing polymerisation.

Contraindications

The product must not be used in patients with known hypersensitivity or allergy to any of the components of the composite (e.g. methacrylate monomers, initiators, photoinitiators or other excipients).

Performance characteristics of the product

Flexural strength ≥ 80 MPa

Application

1. Clean the tooth before preparation and remove all cleaning residues with water spray. Determine tooth colour. Use of a rubber dam is recommended.
2. Prepare the cavity according to the general rules of adhesive technique. Bevel the enamel margins and protect areas close to the pulp.
3. If necessary, etch using your preferred technique, e.g. Total Etch. Follow the instructions for use of the material.
4. Apply bonding agent.
5. If necessary, use an underfilling of thin composite or compomer. Use of a mould is recommended.
6. Fill the cavity in the desired colour in layers no thicker than 2 mm. Light-cure each layer for at least 20 seconds. The light device should emit light of 450 nm. The light intensity should be at least ≥ 600 mW/cm². Bring the light source as close as possible to the layer to be polymerised.
7. Surplus removal, contouring and fine finishing can be carried out with diamond finishers and flexible pads of decreasing grit size, for example. High-gloss polishing can be carried out with polishing pastes using polishing cups/discs.

Precautions

The product is intended exclusively for dental use.

Avoid direct skin or eye contact with uncured material. In case of contact, rinse immediately with plenty of water.

Uncured composite may cause skin irritation and allergic reactions.

Suitable protective gloves and safety goggles must be worn.

Before application, make sure that the cavity is dry and free of saliva, blood and oil to ensure optimum adhesion.

Ensure that the light is of sufficient intensity (as specified by the manufacturer).

The material must not be applied in excessively thick layers. The maximum layer thickness per polymerisation cycle recommended by the manufacturer must be observed in order to ensure complete curing.

The product must be stored in accordance with the storage instructions (temperature, protection from light). Direct sunlight and excessive heat can impair workability and curing.

Side effects

In rare cases, local reactions may occur in sensitive or sensitised persons, in particular:

hypersensitivity reactions of the skin or mucous membrane to methacrylates, post-operative sensitivities (e.g. short-term sensitivity to temperature or pressure), allergic contact dermatitis in dental personnel in the case of repeated skin contact with non-polymerised composite.

If hypersensitivity reactions occur, discontinue treatment immediately and remove the material. The affected area should be rinsed with water and the symptoms treated as necessary.

Should you become aware of any undesirable side effects – even if you are uncertain about the cause – please always be sure to notify us, describing the accompanying circumstances and symptoms as precisely as possible.

Interactions

Contact with materials containing eugenol (e.g. zinc oxide-eugenol cements) can inhibit polymerisation of the composite and should be avoided.

Residues of cleaning, disinfection or impression materials may impair the bond between the composite and the adhesive system. Before application, ensure that the tooth surface has been completely cleaned and dried.

Excess moisture, blood or saliva during application lead to adhesion problems and can promote the formation of marginal gaps and secondary caries.

Storage

Competence Universal should be stored in a dry place, protected from light and at a temperature between 0°C and 22°C.

Minimum shelf life

Do not use after the expiry date.

Variants

- [REF]** WP4250: Competence Universal, set of 4 syringes of 4.5 g each in various Vita® colours (A2, A3, A3.5, B2), 10 ml C-Bond, 2 ml Extra-Gel, 2 ml Glass-Liner + accessories
- [REF]** WP4251: Competence Universal, set of 4 syringes of 4.5 g each in various Vita® colours (A2, A3, A3.5, C2), 10 ml C-Bond, 2 ml Extra-Gel, 2 ml Glass-Liner + accessories
- [REF]** WP4346: Competence Universal, set of 4 syringes of 4.5 g each in various Vita® colours (A2, A3, A3.5, A3 opaque), 10 ml C-Bond, 2 ml Extra-Gel, 2 ml Glass-Liner + accessories
- [REF]** WP4388: Competence Universal, set of 4 syringes of 4.5 g each in various Vita® colours (A2, A3, A3.5, A2 opaque), 5 ml C-Bond, 2 ml Extra-Gel, 2 ml Glass-Liner + accessories
- [REF]** WP4252: Competence Universal, refill, A1, 4.5g
- [REF]** WP4253: Competence Universal, refill, A2, 4.5g
- [REF]** WP4254: Competence Universal, refill, A3, 4.5g
- [REF]** WP4255: Competence Universal, refill, A3.5, 4.5g
- [REF]** WP4088: Competence Universal, refill, A4, 4.5g
- [REF]** WP4294: Competence Universal, refill, B1, 4.5g
- [REF]** WP4256: Competence Universal, refill, B2, 4.5g
- [REF]** WP4264: Competence Universal, refill, B3, 4.5g
- [REF]** WP4257: Competence Universal, refill, C2, 4.5g
- [REF]** WP4266: Competence Universal, refill, C3, 4.5g
- [REF]** WP4265: Competence Universal, refill, D2, 4.5g
- [REF]** WP4258: Competence Universal, refill, D3, 4.5g
- [REF]** WP4260: Competence Universal, refill, A2 opaque, 4.5g
- [REF]** WP4259: Competence Universal, refill, A3 opaque, 4.5g
- [REF]** WP4261: Competence Universal, refill, Incisal, 4.5g
- [REF]** WP4267: Competence Universal, refill, B2 opaque, 4.5g
- [REF]** WP9055: Competence Universal, refill, BXL, 4.5g
- [REF]** WP4349: Competence Universal, set, 1 x A3, 4.5g, 1 x 10 ml C-Bond
- [REF]** WP4397: Competence Universal, refill, Bleach, 4.5g
- [REF]** WP4391: Competence Universal, refill, OM1, 4.5g
- [REF]** WP4392: Competence Universal, refill, OM2, 4.5g
- [REF]** WP4293: Competence Universal, refill, OM3, 4.5g

Disposal

Unused material can be disposed of with household waste after curing.

Processing instructions

The processing instructions and precautionary measures described in the instructions for use supplied with the product must always be observed. If an adverse reaction occurs in the patient or other adverse events arise despite following these instructions, please always be sure to notify us, describing the circumstances as precisely as possible.

Any serious incidents relating to the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is based.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

H317 May cause allergic skin reactions

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5 ml C-Bond, 2 ml Extra-Gel, 2 ml Glass-Liner + accesorios

[REF] WP4252: Competence universal, recambio, A1, 4,5g

[REF] WP4253: Competence universal, recambio, A2, 4,5g

[REF] WP4254: Competence universal, recambio, A3, 4,5g

[REF] WP4255: Competence universal, recambio, A3.5, 4,5g

[REF] WP4088: Competence universal, recambio, A4, 4,5g

[REF] WP4294: Competence universal, recambio, B1, 4,5g

[REF] WP4256: Competence universal, recambio, B2, 4,5g

[REF] WP4264: Competence universal, recambio, B3, 4,5g

[REF] WP4257: Competence universal, recambio, C2, 4,5g

[REF] WP4266: Competence universal, recambio, C3, 4,5g

[REF] WP4265: Competence universal, recambio, D2, 4,5g

[REF] WP4258: Competence universal, recambio, D3, 4,5g

[REF] WP4260: Competence universal, recambio, A2, 4,5g

[REF] WP4259: Competence universal, recambio, A3, 4,5g

[REF] WP4261: Competence universal, recambio, Incisal, 4,5g

[REF] WP4267: Competence universal, recambio, B2, 4,5g

[REF] WP9055: Competence universal, recambio, BXL, 4,5g

[REF] WP4349: Competence universal, juego, 1 x A3, 4,5 g, 1 x 10 ml C-Bond

[REF] WP4397: Competence universal, recambio, Bleach, 4,5g

[REF] WP4391: Competence universal, recambio, 0M1, 4,5g

[REF] WP4392: Competence universal, recambio, 0M2, 4,5g

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