Calcident LC[®]

- Seite 2 - de: Gebrauchsanweisung
- Page 3 - en: Instructions for use
- Page 4 - fr: Manuel d'instruction
- Página 5 - es: Instrucciones de uso
- Pagina 6 - it: Istruzioni per l'uso
- Strana 7 - bs: Uputstvo za upotrebu
- Pagină 8 - ro: Instrucțiuni de utilizare
- Sayfa 9 - tr: Kullanma kılavuzu
- Strona 10 - pl: Instrukcja użytkowania
- Стр. 11 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist

Composition

Qualitative information according to the information regarding biocompatibility:
Calcium hydroxide, barium sulphate, oligomeric methacrylates, HEDMA

Quantitative information: Substances (quantity according to biocompatibility):

Oligomeric methacrylates and thinner > 70%
Calcium hydroxide > 4%,
Barium sulphate > 4%

Intended purpose of the product

Calcident LC is a light-curing, radiopaque, one-component material with calcium hydroxide.

Indications

- Indirect pulp capping
- Underfilling under any filling material
- Acid protection when using the etching technique

Contraindications

Do not use on patients with a known allergy to methacrylic acid derivatives.

Performance characteristics of the product

Indirect protection of the pulp by calcium hydroxide.

Clinical benefits

Indirect protection of the pulp against acid attacks by underfilling/filling materials.
Calcident LC is a paste that is easy to apply and can be polymerised with light. The compound is radiopaque.

Application

When applying, ensure that the tooth surfaces are completely dry and avoid contamination.

Preparation is in line with the rules of applied filling therapy. Apply an aqueous calcium hydroxide preparation in the immediate vicinity of the pulp. Permanent isolation is recommended to prevent marginal gaps and bacterial infiltration in cervical/proximal margins. Calcident LC can be applied directly from the syringe.

For underfillings with a thickness of more than 1 mm, apply in layers and polymerise.

Cure Calcident LC with LED light, bringing the light source as close as possible to the material (approx. 1-2mm) and curing for at least 40-60s. Because of its strength, excess material cannot be removed with a probe but only with a rotating instrument.

Place fillings according to the manufacturer's instructions.

Please note

The halogen or LED light curing device should emit at 450 nm with an illumination intensity ≥ 600 mW/cm² and must be checked regularly. Place the light as close as possible to Calcident LC.

Warnings/Residual risks**Notes:**

Calcident LC contains calcium hydroxide, TEGDMA, BHT, UDMA and amine and should therefore not be used on patients with hypersensitivity to these ingredients. Calcident LC adheres to materials such as K + B materials, methacrylate-based temporary sealants and certain impression materials, e.g. polyether. This can lead to removal of the underfilling. In these cases, we recommend applying a release agent such as Vaseline or glycerine, which should be removed again before final filling. Eugenol and other impurities (e.g. thymol) will cause incomplete curing. Contact with these materials should be avoided. Exposure times that are too short lead to incomplete curing, so please check the curing lamp and the light guide regularly.

The application aids (cannulas, disposable brushes, etc.) included with the product are intended for single use on one patient only.

Side effects

No side effects are known to date. In individual cases, hypersensitivity to components of the material cannot be ruled out.

Avoid skin contact with the paste.

In case of accidental contact, wash immediately with soap and water. In case of eye contact, rinse immediately with plenty of water and seek medical advice.

Should you become aware of any undesirable side effects – even if you are uncertain about the cause – please always be sure to notify us, describing the accompanying circumstances and symptoms as precisely as possible.

There are no known risks in the event of an overdose.

Interactions

There is no known interaction of the product with other medical products, pharmaceuticals or other substances.

Storage

Always store closed in the refrigerator (approx. +2°C to +8°C) and protected from light and moisture.

Minimum shelf life

Do not use after the expiry date.

Variants

 WP4271 Calcident LC 1 x 2ml

 WP4270 Calcident LC 2 x 2ml

Disposal

Unused material can be disposed of with household waste after curing.

Processing instructions

The processing instructions and precautionary measures in the package leaflet accompanying the product must be observed in all cases. If an adverse reaction occurs in the patient or other adverse events arise despite following these instructions, please always be sure to notify us, describing the circumstances and symptoms as precisely as possible.

Any serious incidents relating to the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is based.

We know how

 **dent a pharm**
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Attention

H317 - May cause an allergic skin reaction

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