D

# Vision Prophy Paste®

## Gebrauchsinformation

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# Vision Prophy Paste®

## Instruction for use

Vision Prophy paste is a universal polishing paste based on Pumice powder. By its high RDA value (Radioactive Dentin abrasion) of 237 it has brilliant polishing characteristics. Vision Prophy paste is applicable inside, as well as outside of the mouth. Vision Prophy paste does not contain fluoride, because fluoride can damage the bleaching effect.

### Indications

- Cleaning before bleaching
- Professional polishing of teeth
- Polishing of plastics and precious metals
- Polishing of crowns and filling materials

### Application

Open the box and remove the necessary quantity of paste. Close the box immediately. Apply a little quantity of paste on a brush or polishing cup. Do not mix with water. Give the paste on tooth surfaces or on the materials you want to polish. Work low-speed if it is possible. After the polishing patients have to rinse out thoroughly with water. Clean polished materials with water.

### Contra-indications

Not for use at patients who are known to be allergic.

### Side effects

So far no side effects are well-known. In case of eye contact, immediately rinse with plenty of water. If you experience any undesired effects, even in case of doubt, we ask for notification in every case, describing the concomitants and symptoms as detailed as possible.

### Processing instructions

See under point "application".

### Warnings/precautions

It is recommended to the patient and the dentist to carry an eye protector. Product use is only indicated for the indication ranges. In case of a preparation for tooth whitening (Bleaching) no paste should remain on the tooth surfaces. This can impair the bleaching effect.

### Storage

Keep well locked at room temperature (or lower). If sedimentation should happen, or the paste will be drained, then that does

not impair the effectiveness of the product.

### Packaging

- REF** WP6120 Vision Prophy Paste cinnamon flavoured 1 x 160 g
- REF** WP6139 Vision Prophy Paste strawberry flavoured 1 x 160 g
- REF** WP6140 Vision Prophy Paste lemon flavoured 1 x 160 g

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Willmann & Pein GmbH