BH

Calcident 450[®]

Uputstvo za upotrebu

Sadržaj:

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Composition

100 g Calcident contains:
45 g Calciumhydroxide and 5 g Bariumsulfate.

Indications

Supply of the caries-free, pulp-near dentine, treatment of caries profunda, indirect pulp-capping, direct pulp-capping, pulpotomy, inlay in root-canal.

Contra-indication

The use of Calcident 450 is contraindicated: - opened pulp/-exposure in carious dentine, at Ignited modified pulp, at super-sensitivity against one of the ingredients.

Particular precautions

Calcident 450 is always to be stored well closed. While application do not put higher pressure on opened pulp.

Calcium hydroxide causes local irritation and corrosive effects of the skin, eyes and mucous membranes. particles of calcium hydroxide in contact with the skin or eyes must be mechanically removed by appropriate meas, then rinsed immediately and thoroughly with water. Do not use this medical product after expiry date. Keep this m edicine out of reach of children.

Interactions

The harmful effects of with acid mixed cements onto the pulp-/dentin-system can be reduced throughout calciumhydroxide-applications.

Method of use

The given quantity of Calcident 450 requires to the particular range of application:

- Caries profunda-treatment / indirect. Pulp-capping – Calcident 450 will pressure- less be applied in a thin layer onto the bottom of cavity. Thereafter a temporary-/ or final filling will be set.
- Direct pulp-capping – After drying of the working-area Calcident 450 will be applied and a suitable cover-filling is closing the tooth finally.
- Vitalamputation – Calcident 450 is builds good conditions for healing of remaining pulp-stumps throughout the accepted local bounded etch-effects of the calciumhydroxide to the remaining pulp.
- Lining/pulp-protection – The space- saving and radiopaque Calcident 450 builds a

GB

Calcident 450[®]

Instructions for use

pulp-protection even in front-teeth-area under Silicate- or composite- fillings a thin, whitish insulating-layer against cement-acids or free monomer.

5. Dentine-Wound-bandage for grinded vital teeth – With a brush or a wad of cotton wool Calcident 450 will be applied onto the preserved tooth-stump and carefully dried with warm air. Avoid contact with the gingiva Beforehand dip the brush into cooked water/Aqua dest. to dilute the Paste.
6. Medical Inlay into infected root-canals – Each root-canal should be air-bubble-free filled with Calcident 450. Eventually the inlay should be relaced after 6 to 8 weeks.

Side-effects

Side-effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side-effects - even in case of doubt - then please contact us and give a precise description of the circumstances and symptoms.

Working instructions

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please infarm us and give a precise description of the circumstances.

Storage

Immediately after use the syringe top has to be protected with a cap. In case of direct application the tip has to be sterilised and disposed. Do not use hardened paste. Do not store less than + 8° C! Do not use after expiry date.

NOTE

Applicator-tools (tips, brushes) are only determinate for single-use at only one patient.

Presentation

WP4200 1 syringe with 2.0 ml paste + tips
 WP4201 3 syringes with 2.0 ml paste + tips

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