Premium Putty®

Gebrauchsinformation

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Premium Putty®

Instruction for use

Kneadable silicone based precision impression material (Polysiloxane-condensation curing) High viscous for dental practices (soft).

IN accordance with DIN EN 4823 Type 0, ISO 4823, Type 1 Cat B

Color: yellow

Application

- Initial impression for the double impression technique
- Situation Impression: crush bite
- Impression-material for the sandwich-technique
- mixing: manually

Processing:

1 Measuring-spoon kneading silicone soft - 2,5cm paste-hardener

Mixing-time: 30 seconds

Total working-time for soft and paste-hardener - 2 minutes

Setting-time: 3 Minutes - statement for 23°C (73,4°F), rel. humidity

Shore-hardness: approx. 60 (soft)

Higher dosages of Universal-paste hardener and higher temperatures accelerate setting; lower dosages of universal-hardener and lower temperatures delay setting. Impression preservation: store at room-temperature under dry conditions.

Model preparation: Pouring of the impression should be carried-out not sooner than 15 minutes after removal of the impression from the mouth.

Electroplating: The silicone material is compatible with the customary dental acid or respectively, alkaline copper or silver electroplating baths.

Handling and safety measures:

Do not allow the universal hardener to come into contact with the eyes, if it does, rinse eyes immediately and thoroughly with clean water and go directly to an oculist.

Avoid getting traces of the impression mass on clothing as chemically cross-linked elastomer impression material are chemically resistant. Close container carefully after use.

Storage: Store not above 25°C (77°F)

Shelf-life: See use-by date on pack

The preparation, usage and application of the product supplied lies exclusively in your area of responsibility since we have no opportunity of exercising control over these matters, in so far as we give advice on technical application matters verbally, in writing or through trials, then this advice is to be understood as being nonbinding information same applies in respect of the possible protective rights of third parties. Such advice does not release you from the obligation of testing the product supplied for their suitability for the intended purpose and process. Our liability is in every case of damages limited to the value of the materials supplied by ourselves and the goods utilized by yourself. The paper quality of our products is guaranteed in accordance with the terms of our general conditions for sale and delivery.

Packaging

WP4324 Premium Putty soft - 900ml
WP4326 Premium Activator-paste - 60ml
WP4325 Premium Light "wash" - 140ml