D

WP Metalprime®

Gebrauchsinformation

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Adhesions liquid for the connection of metal with acrylic and composite resins.

Indication

WP Metalprime provides a better adhesion of acrylic and composite resins on all kinds of metal.

Application

Sandblast metal surface with aluminium oxide and blow clean with air gun.(recommended particle size 110 with 2,8 bar) Please avoid contamination with separating agents, oil or moisture.

Put 2-3 drops of the WP Metalprime into a suitable container. Apply a thin layer onto the adhesive surface with the provided brush and let it dry. If required a second layer can be applied.

Apply the resin on the metal immediately after surface treatment. Do not use the provided brush for different products, only for the WP Metalprime.

Application range

- Repair of dentures Clasp, metal framework, attachment Roughen the metal surface with a carborundum grinder and prepare the surface as described above. Build up the self-cure resin in conventional manner.

- Opaque resin to metal Composite crowns and composite veneers Prepare the surface as described above and apply opaque resin in conventional manner. Polymerize the material according to the manufacturer's recommendation.

- Resin cement to metal for metal inlays, crowns and bridges Prepare the surface as described above and bond the restoration with resin cement.

- Connection of resins with metal orthodontic appliances Prepare the surface as described above and apply the orthodontic resin in conventional manner on the appliance.

- Denture base magnetic attachment Prepare the surface as described above and append attachment in conventional manner with self curing acrylic resin.

- Denture base resin to metal base, clasp or attachment Sandblast the adhesive surface with aluminium oxide and blow clean with air gun. Make a wax model from the prosthesis. Clean the adhesive surface after the wax extraction with heated steam. Completely remove the wax residues with a solvent. Afterwards apply WP Metalprime and pack and polymerize denture base in conventional manner.

GB

WP Metalprime®

Instructions for use

• Castings/Veneer-Crowns

Retentions need to be blasted with aluminiumoxide. Apply WP Metalprime with a clean single-use brush and dry by the use of a hot air gun or compressed-air (oilfree) with air-water separator.

Working instructions

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

Contra-Indications

WP Metalprime may only be used from dentists and dental technicians. Containing methacrylate, therefore this product should not be used from / for persons in case of a known allergy to methacrylate. If allergic reactions occur seek medical help immediately

Storage/shelf-life

2 years
WP Metalprime is easily flammable and has to be stored cool and dry. Avoid sources of ignition and direct sunlight. Use product in well ventilated area. WP Metalprime is volatile. Container should be sealed immediately after use.

Presentation

REF|WP5593 2 x 3 ml liquid + dental brushes and brush holder

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