Pro-Fill[®]

Gebrauchsinformation

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Pro-Fill[®]

Directions for Use

Pro-Fill is a light-curing temporary restorative filling material.

Indications

For all types of temporary fillings, temporary protection used for inlay and onlay techniques.

Application

1. Preparation, lining and impression using the standard techniques.
2. Rinse and dry the cavity.
3. Place Pro-Fill in the cavity with a suitable instrument.
4. Cure afterwards for 20 - 40 seconds (according to thickness) with halogen-light. As a rule, 5 mm deep layers are adequately cured in 40 seconds. Increased exposure times are required where the light source cannot be brought close to the material.
5. Check the occlusion. Excess can be cut away with a scalpel. Silicone points are suitable for grinding.

Removal of temporary filling

Push with a rigid instrument into the temporary filling and withdraw.

Side Effects

Undesirable side effects of this medical-product are not to expect if working corresponding to the instruction of use.

In case of side effects - even when doubtful - please inform us with exact description of accompanying circumstances and symptoms.

Interaction

Pro-Fill may adhere to synthetic dental materials, and in such cases a separating medium is indicated. Do not use liners containing eugenol as they may impede Pro-Fill's setting.

Contra-indications

Do not use in known cases of allergy to acrylates. No side effects are recorded up to date.

Instruction references

Take care of all working- and caution advices. In case of unexpected side effects or other reactions (even if work was made to the instruction of use) please inform us, if possible, with exact description of accompanying circumstances and symptoms.

Active ingredients

1 g contains 229 mg silicone dioxide, 595 mg polyesterurethane acrylate and 175 mg triethylen-glycoldimethacrylate.

Storage

Store at room temperature up to 21° C. If not in use for some times, cool storage is advised. Keep away from light. Do not use after expiration date.

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Willmann & Pein GmbH