BH

P-CEM Automix®

Uputstvo za upotrebu

P-Cem Automix je privremeni cement koji ne sadrži eugenol i koji ne sprečava stvrdnjavanje materijala za ispunu i učvršćivanje na bazi plastičnih materijala. P-Cem Automix je indiciran kod alergija na eugenol.

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P-Cem Automix upotrebljava se kao provizorni cement za krunice i mostove, kao i za zatvaranje.

Bitne komponente

Smolne kiseline i metalni oksidi.

Interakcije

Nisu poznate.

Upozorenja

Izbjegavajte kontakt zamiješanog cementa s kožom ili sluznicom. Ako ipak dođe do kontakta, uklonite materijal vatom natopljenom alkoholom i isperite s dosta vode. U slučaju kontakta s očima, oči širom otvorenih kapaka isperite s dosta vode i konzultirajte oftalmologa.

Napomene

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P-CEM Automix®

Instructions for use

P-Cem is an eugenol-free temporary attachment cement, which contains calciumhydroxide and hydroxylapatit. P-Cem is used for fitting temporary crowns, bridges and for temporary sealing purposes. P-Cem is supplied in an automix cartridge together with mixing cannulas which will guarantee a correct economical mixing ratio at all times. The P-Cem automix cartridge can be used with the majority of mixing guns on the dental market.

Method of use

- Place the automix cartridge in the mixing gun.
- Remove the closure from the cartridge and verify that the exit of the two materials is not obstructed.
- Press the trigger of the mixing gun and check that the two components are exiting the nozzle at the same rate.
- Attach the mixing cannula.
- Apply a small amount of P-Cem directly onto the temporary crown or bridge.
- After use leave the mixing cannula in place, it will protect the nozzle until the next application.

Working time: approx. 1:30 min.
Curing time in the mouth: 2–3 min.

Side effects

Undesirable side-effects are extremely rare if the product is prepared and used properly. Should you become aware of any undesirable side-effects, including in cases of doubt, please notify us, giving as precise a description of the attendant circumstances and symptoms as possible.

Working instructions

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

Supplementary instructions

Do not use P-Cem on dry cores, moisten them with some water beforehand. Residues of P-Cem can be removed easily after spraying them shortly with a cooling spray.

Storage

P-Cem Automix should be stored at room temperature and not above 25 °C. Do not use after the expiry date. See label for shelf life and expiry date. Store the cartridge with used syringe until the next use.

Presentation

REFWP4015 1 cartridge P-Cem, 25 ml + mixing cannulas
REFWP4016 2 cartridge P-Cem, 25 ml + mixing cannulas

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