# C-Bond Self-Etch®

## Gebrauchsinformation

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# C-Bond Self-Etch®

## Instruction for use

C-Bond Self Etch is a self-etching, light curing one component adhesive for the adhesive restoration technique.

**Composition**  
C-Bond Self-Etch contains methacrylate, photoinitiators.

### Indications for use

Adhesive for the adhesive restoration technique. - Fixing of composite-material and compomer at enamel and dentin

- adhesive fixing of restorations of ceramic- or composite at enamel and dentin  
- Fixing of composites at ceramic, compositematerial, metal or amalgam  
- also usable as dentin-sealer

### Side effects

Side effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side-effects - even in case of doubt - then please contact us and give a precise description of the circumstances and symptoms. Constituent parts of C-Bond Self-Etch can lead to sensitivity or an allergic reaction with a predisposed person.

### Working instruction

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions have been observed, then please inform us and give a precise description of the circumstances.

### Interactions

Phenolics and other substances (e.g., zinc-oxide-eugenol), which may effect the polymerisation, must not be contacted with C-Bond Self Etch.

### 1. Conventional filling-technique

**a) Preparation of the cavity**  
According to the principles of the restoring-technique. If necessary a punctional capping of the pulp with a strong-setting calciumhydroxide-cement is to be set. Conditioning of the enamel with a finishing bur. Unpolished enamel is to be etched with etching-gel or cleaned with pumice. Afterwards rinse enamel well and dry it. The use of dam is recommended for hygienic reasons.

### b) Application

Shake bottle well before use and give one drop into a dispensing well. Apply gently C-Bond Self-Etch for 20 s to dentin and enamel. Dry gently with oil-free air the C-Bond Self Etch for 5 s.

Afterwards light-cure for 10 s (halogene- or LED-polymerisation-device with a light-intensity of >800 mW/cm<sup>2</sup>). If the light-intensity is low the light-curing prolongs correspondingly.

**c) Fillings of composite and compomer**  
Use restoration material according to the instruction of the manufacturer.

**2. Fixing of inlays/onlays, veneers and crowns**  
While using C-Bond Self-Etch it is to observe that enough light (according to the instructions of the manufacturer) is getting into the complete restoration.

**a) Preparing of the restoration**  
Prepare the restoration concerning to the working instructions of the corresponding substance..

**b) Preparation of the cavity**  
As described in point 1.1.

### c) Application

As described in point 1.2.

### d) Cementation

According to the instructions of the manufacturer.

### 3. Adhesion of other surfaces

#### a) Preparation

#### b) Ceramics

**Intraoral application:**  
Ceramic surfaces should be roughened with a diamond bur, sandblasted or etched with 37% phosphoric-acid (z. B. Extra Gel), rinsed and dried with oil-free air.

#### Extra oral application:

Ceramic surfaces can be etched with 5% HF acid afterwards rinsed and dried with oil-free air.

#### Optional:

Silanization of ceramic-surfaces (according to instructions of manufacturer)

#### c) Metal and Amalgam

Roughen surfaces with a diamond bur or sandblast them.

Afterwards clean with stream jet (extra oral) or acetone and dry with oil free air.

#### d) Composite

Surfaces of composites are to be roughened with a disposable brush for 20 s onto the surface. Dry the bonding gently for 5s with oil free air. Afterwards light-cure for 10 s (halogene- or LED-Polymerisation-device with a light-intensity of >800mW/cm<sup>2</sup>). If the light-intensity is lower the light-curing prolongs correspondingly.

#### e) Application

Shake bottle well before use and give one drop into the dispensing well. Close the bottle immediately after use. Apply C-Bond Self Etch with a disposable brush for 20 s onto the surface. Dry the bonding gently for 5s with oil free air.

#### f) Restoration

Use the restoration-material according to the instructions of the manufacturer.

#### 4. Sealing of the dentin

Please take note of the working instruction at point 1.1 and 1.2.

#### Emergency-measures

Acute danger at the patient throughout accidentally swallowing is in consideration of the small quantities (application with disposable brush) not to be expected. If there is a direct contact with the oral mucosa rinse carefully with tap water. In case of eye-contact rinse carefully with tap water. For evaluation and after treatment contact an ophthalmologist.

#### Toxicological effect

The toxicity of the direct the vital dentin contacting and accordingly used C-Bond Self-Etch which penetrates the dentin canals is comparable with glasionomer cement or polycarboxylat.

#### Note

Only for the use of dentists or dental technicians or people who are assigned of them. Keep away from children! Dispose only residue-less packaging.

#### Storage and marking

Expiry-date and LOT - see package.

#### Storage

0°C - 22°C  
Keep away from direct sunlight and heat sources.

#### Hygienic recommendations

Adherence of the following recommendations assures optimal hygienic terms while using C-Bond Self-Etch.

- Disposable brushes should only be used at one patient

- Close bottle immediately after use and disinfect with current agents.

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