I

Competence universal®

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Competence universal[®]

Instructions for use

Competence universal is a light curing, x-ray opaque universal filling composite for restorations of all cavity classes. Its special modeling properties make it particularly suited for the multiple shade layer technique. This not only guarantees a natural and highly aesthetic filling, but also minimisation of polymerisation shrinkage.

Indication

Restoration of all cavity classes

Recommended usage

- Prior to preparation, clean the tooth, remove all clearing residues with water spray. Determine the Vita® tooth shade. Use a rubber-dam where possible.
- Prepare the cavity according to the general rules of adhesive technology, round off enamel edges and protect pulp areas.
- Etch as required, according to your preferred technique, e.g. Total-etch. Follow the instructions for use of the materials used.
- Apply bond adhesive, (e.g. C-Bond)
- Apply a lining of thin flowing composite (Competence Flow) or compomer as required. We recommend using a matrix.
- Fill the cavity in the required shade in a maximum 2 mm thickness. Light-cure each layer for 20 seconds.
- Surplus removal, contouring and fine shaping are carried out e.g. with a diamond finisher and flexible discs of decreasing grain size. High-gloss polishing can be carried out with a polishing paste (e.g. Vision Paste) using polishing wools/discs.

Caution

- Light devices should emit at 450 nm and be regularly inspected. The light should be placed as close as possible to the filling material.
- Do not remove the oxygen-inhibited layer, as this is required for the bond with the next layer.
- Follow the instructions for use of the other materials used with Competence universal.

Interactions

Avoid materials containing eugenol, moisture and airborne oil – they can prevent the polymerisation of the contact area. Use of cationic mouth rinses, plaque revealators or chlorhexidine can cause discoloration.

Contraindications

The application of Competence universal is contraindicated if drainage or the recommended application technique cannot be achieved. Do not apply directly on open pulp or if there are any known allergies against one of the components.

Side effects

No side effects are known to date. Hypersensitivity against components of the material cannot be excluded in every case. Avoid skin contact with the paste. In the event of inadvertent contact, wash immediately with soap and water. In the event of eye contact, immediately rinse with plenty of water and seek medical treatment. If you experience any undesired effects, even in case of doubt, we ask for notification in every case, describing the concomitants and symptoms as detailed as possible.

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