# WP PerfectPrint -flexibel<sup>®</sup>

## für Modelle

# WP PerfectPrint -flexible<sup>®</sup>

## for Models

### Gebrauchsinformation

#### Allgemeine Informationen

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### Instructions for Use

#### General information

WP PerfectPrint -flexible- is a methacrylate-photoinitiators, pigments and additives. based light-curing resin for use in the dental laboratory. For SLA 3D printers operating at a wavelength of 405 nm.

#### Intended use

Production of dental models for 3D printing technology.

#### Special precautions

- Non-compliance with instructions and improper use may lead to impairment of quality.
- Wear protective gloves, protective clothing and safety goggles while working.
- Avoid contact with eyes! If liquid material comes into contact with eyes inadvertently, rinse immediately with plenty of water and consult a doctor if necessary.
- Avoid direct skin contact with liquid material and components prior to post-curing.
- Keep out of reach of children.
- Observe safety data sheet.

#### Side effects

- Irritation of eyes and skin.
- The polymerised material should not be used if there are allergies to any of the ingredients or in case of contact allergies.
- In rare cases, allergic reactions to components of WP PerfectPrint-flexible- may occur. Inhalation of monomer vapour and skin contact with non-polymerised materials should be avoided.

#### Notes on use

- Shake the bottle for at least one minute before use.
- When filling the pan with resin, observe the instructions of the device manufacturer.
- Note the temperature that the device manufacturer specifies for production. The resin can also be processed at room temperature.
- Darken the room slightly before removing the print platform with the printed object. Then, open the hood and remove the printing platform with the printed object.
- Remove the printed object from the platform.
- Only remove the support structures after cleaning and post-curing.

#### Using used resin

- After sieving (removal of polymerised particles) with a sieve gauze (about 200 µm), the used resin can be mixed with fresh resin 1:1 and reused.

#### Post-processing of the printed object

- Clean the printed object with ethanol or with a soft brush for a maximum of 3 minutes.
- Optional: Clean printed object in a separate container with fresh ethanol in an ultrasonic bath as needed (5-10 mins).
- Optional: Clean openings, holes and gaps with compressed air.

#### Post-exposure of the printed object

It is recommended to post-cure the printed object on each side for 20 minutes in a UVA light-curing device with 4 lamps of 9 watts each. Please note the information provided by the device manufacturer.

#### Restrictions on use

- Intended exclusively for the creation of dental models in the dental sector.
- Must not be brought into direct contact with the patient.

#### Composition

Mixture of acrylate and methacrylate resins, photoinitiators, pigments and additives.

#### Storage and durability

- Store at room temperature/0-22 °C in a dark, dry place.
- Only handle WP PerfectPrint -flexible- as briefly as possible to reduce exposure to daylight/room light. Excessive exposure to light leads to an unwanted hardening of the material.
- Close the container immediately after each use and keep it tightly closed.
- Do not use after the expiration date!

#### Packaging

- WP5145 WP PerfectPrint -flexible-, white, 1000 g
- WP5146 WP PerfectPrint -flexible-, white, 500 g
- WP5148 WP PerfectPrint -flexible-, pink, 1000 g
- WP5149 WP PerfectPrint -flexible-, pink, 500 g
- WP5151 WP PerfectPrint -flexible-, yellow, 1000 g
- WP5152 WP PerfectPrint -flexible-, yellow, 500 g
- WP5153 WP PerfectPrint -flexible-, grey, 1000 g
- WP5154 WP PerfectPrint -flexible-, grey, 500 g
- WP5155 WP PerfectPrint -flexible-, blue, 1000 g
- WP5156 WP PerfectPrint -flexible-, blue, 500 g
- WP5157 WP PerfectPrint -flexible-, black, 1000 g
- WP5158 WP PerfectPrint -flexible-, black, 500 g
- WP5163 WP PerfectPrint -flexible-, beige, 1000 g
- WP5164 WP PerfectPrint -flexible-, beige, 500 g
- WP5170 WP PerfectPrint -flexible-, yellow opaque, 1000 g